



**Therapist
Support
Laboratory**

52 Nicholson St, Abbotsford, Vic 3067

PO Box 1022 Collingwood, Victoria, 3066

Tel:03 9421 2880 Fax: 03 9421 1335

Website: www.tsaustralia.com.au

Email: tsl@tsaustralia.com.au

Stocking Assist



Step One

Slide the Stocking Assist over the patients foot.

Fold Stocking Assist inside itself as illustrated. 



Step Two

Apply compression stocking, pulling the stocking over the foot up to the heel.



Step Five

Toes **should not** be visible when fitted correctly.



Step Three

Fit the stocking over the calf.



Step Four

Grasp the Stocking Assist handle and pull firmly to remove the applicator.

New to TSL



Washing Instructions

Program	Gentle Wash Cycle
Duration	5 - 10 minutes
Chemical	Woolwash detergent pH7
Water Temp	65 - 75 degrees C +
Extract	Slow (industrial only)
Rinse	65 - 75 degrees C +
Spin	Slow (industrial only)
Dry	Warm tumble (not hot) or air dry, short duration



* Avoid the use of alkaline detergents, bleaches and scouring agents

* 'Woolsan' or 'Breathe-Ezy' is recommended for 'Stocking Assist' care.

<u>Sizes Available</u>	<u>Code</u>	<u>Colour</u>
Stocking Assist (standard size)	SAS	(Blue)
Stocking Assist (long - 1400mm length)	SASL	(Red)
Stocking Assist (wide/800mm length)	SAW	(Khaki Green)
Stocking Assist (wide/1400mm length)	SAWL	(Maroon)

* Product colours may vary

